

# clinell<sup>®</sup>

## Universal Wipes 84

---

Simple cleaning  
& disinfection,  
proven in practice

- ✓ Kills 99.99% of bacteria and viruses\*
- ✓ Effective from 10 seconds
- ✓ The UK's number 1 hospital disinfectant wipe brand



# Who are Clinell?

## Trusted nationwide

Since 2006, Clinell has been at the forefront of infection prevention. Today the Clinell Range has a wide selection of products to help clean and disinfect surfaces, but it all started with the Clinell Universal Wipe. Trusted by 9 out of 10 NHS hospitals, Clinell Universal Wipes have become the go-to infection prevention solution within the healthcare industry.

### Clinell Universal Wipes 84



Designed for real-world applications, not just a petri dish, Clinell Universal Wipes kill 99.99% of bacteria and viruses in 30 seconds\*. Perfect for offices, workspaces and vehicles.

### What about contact time?



In our day-to-day lives, we touch lots of things like door handles, equipment & work vehicles, and so do our colleagues. These surfaces are known as high-touch points and can harbour microorganisms which are easily transferred to other surfaces and people.

Clinell Universal Wipes are designed to kill microorganisms dwelling on high-touch points to make your work environments safer for everybody. Fast-acting and simple to use, our Universal Wipes provide a one-step cleaning and disinfection solution you can rely on.

### Why does efficacy matter?



Many different disinfection products show how effectively they remove microorganisms with a percentage on their packaging. This is called a log reduction. Log reductions are calculated by testing disinfectants against different microorganisms and seeing how well they kill them.

For example, if you put disinfectant in a petri dish with 1,000,000 microorganisms and it leaves 1,000 microorganisms, then that disinfectant would have a 3 log rating: equalling 99.9%. However, Clinell Universal Wipes have a 4 log rating, meaning that out of 1,000,000 microorganisms, only 100 remain, resulting in a 99.99% efficacy rating.

# The key to Clinell

## Clinell Universal Wipes 84

Clinell Universal Wipes are designed for real-world applications, not just a petri dish. A lot of science goes into each wipe to provide fast and effective results you can trust, killing 99.99% of bacteria and viruses in 30 seconds\*.



**Kills 99.99% of bacteria and viruses\***




**From 10 second contact time**



**Effective against Coronavirus, Norovirus & *E.coli***


## How to use Clinell Universal Wipes

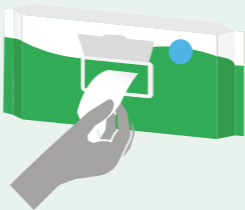

Remove from pack, open out and use flat



Working from top to bottom, clean to dirty, wipe in an 'S' shaped pattern.

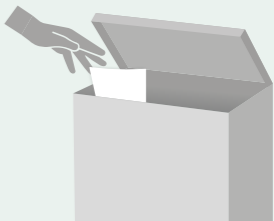

Should your wipe dry out or become soiled, replace it with a new wipe. Let the surface air dry to maximise contact time.

**Clinell Universal 84**  
is made by GAMA Healthcare.  
For more information, speak to your  
GAMA Healthcare Area Manager or visit  
**[www.gamahealthcare.com](http://www.gamahealthcare.com)**

Product	Unit of issue	Order code
Clinell Universal Wipes	Pack of 84 wipes	BCW84

\*Clinell Universal Wipes are proven effective against Adenovirus,  
Norovirus and all known enveloped viruses.

Use biocides safely. Always read the label  
and product information before use.



**GAMA Healthcare Ltd.**, The Maylands Building,  
Maylands Avenue, Hemel Hempstead, HP2 7TG, UK.  
T: +44 (0)20 7993 0030 E: [info@gamahealthcare.com](mailto:info@gamahealthcare.com)  
**[www.gamahealthcare.com](http://www.gamahealthcare.com)**

JBN220448